

INSTRUCTIONS FOR USE


Instrument Disinfectant and Cleaner

Date of revision: 05/03/25

Available in 5 Litre / 2.5 Litre / 1 Litre dispenser bottle




Intended Use / Intended Purpose: Disinfecting and cleaning concentrate for instruments, burs and endodontic instruments and items. Chloride, aldehyde and phenol free with a wide material compatibility and broad microbiological efficacy.

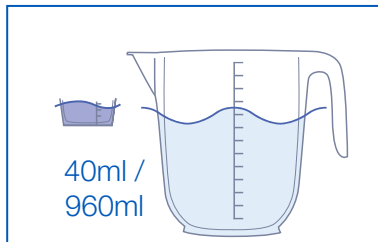


1. Add 20ml $\pm$ 1ml to 980ml of clean water for a 2% solution, 40ml $\pm$ 1ml to 960ml for a 4% solution. To achieve full efficacy use a 4% solution.


The 1L bottle has a built in dosing device and the 2.5L a dosing cap.

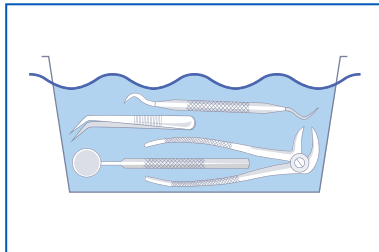

2. Completely immerse the instruments into the solution ensuring that no air is left within hollow instruments.

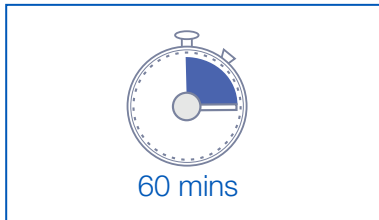

3. To achieve full efficacy use a 4% solution for a 60 minute soaking time.

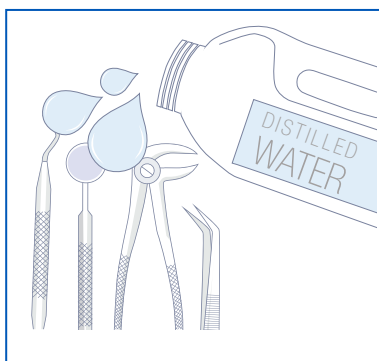

4. To Use: for a 60 minute soaking time. After disinfection, rinse instruments thoroughly with RO or distilled water prior to further processing and sterilisation. Follow the reprocessing instructions from the instrument manufacturer. Change the solution daily. See product labels and MSDS for further information.

Indications / Efficacy: Efficacy 2%: Bactericidal including MRSA (15 min). Yeastcidal (15 min). All enveloped viruses (15 min) to include: Vaccinia virus, HBV, HCV, HIV, Hepatitis Delta virus, Influenza virus, Coronavirus, Herpesviridae, Filoviridae (Ebola), Rubella, Rabies, Measles, Poxviridae. Efficacy 4%: As above and to include Non-enveloped viruses (30 min): Murine norovirus, Adenovirus 5. Mycobactericidal & Tuberculocidal (60 min).

Tested to Standards: EN17111, EN14885, EN14561, EN14562, EN14563, EN14476, EN14348, EN13727, EN13624.

Composition: Aqua, N-(3-Aminopropyl)-N-Dodecylpropane-1,3-Diamine (4.1% w/w 2372-82-9), DMPAP (3.5% w/w), Alcohols, C9-11, Ethoxylated, Tetrasodium N,N-Bis(Carboxylatomethyl)-L-Glutamate, Monoethanolamine, Corrosion Inhibitor, Perfume contains CINEOLE - Cineols - Eucalytus; HYDRATROPALDEHYDE; TRANS-MENTHONE; p-menth-1-en-4-ol., Dye.

Storage. Store in a well-ventilated place. Keep container tightly closed. Keep cool.

Contraindications. Heat.

Disposal. Do not discharge concentrate into drains or rivers.

Intended Users. For professional use only. The products are used within the dental surgery by the dentist or dental assistant as part of their general disinfecting routines.

Patient Population: Dental surgery patients, both adults and children.

Any serious incident related to this medical device must be reported to the manufacturer and the competent authorities.

Danger. H315: Causes skin irritation. H318: Causes serious eye damage. H410: Very toxic to aquatic life with long lasting effects. P264: Wash hands thoroughly after handling. P280: Wear protective gloves, eye protection, protective clothing. P305+P351+P338+P310: IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Immediately call a doctor. P332+P313: If skin irritation occurs: Get medical advice/attention. P362+P364: Take off contaminated clothing and wash it before reuse. P391: Collect spillage.



Danger.



UFI: 3X00-W0GP-900K-VWW7

CE
1639



Manufactured in the UK by:
Topdental (Products) Ltd.
Unit 1, Holmfield Industrial Estate,
Halifax, W. Yorks, HX2 9TN. UK
Tel +44 (0)1535 652750
www.topdental.co.uk
www.virosafeuk.com
Email: info@virosafe.com