



VIROsafe

365daysafe

DAILY ASPIRATOR CLEANER AND DISINFECTANT

- **Effective in only 15 mins**
- **Economical 2% dilution ratio**
- **Low foaming**
- **Contains EDTA to remove calcium**

2 in 1
cleans and
disinfects

15
minute



Concentrated disinfectant and cleaning solution safe to use on all types of dental aspirator / suction system.

An all in one formula is designed for to use everyday, replacing the need for additional cleaning solutions.

The unique low foaming solution ensures no risks to sensitive equipment.

The dental suction system is one of the most heavily used and contamination-prone components in any clinic. Keeping it clean and disinfected is essential for maintaining efficiency, hygiene, and patient safety. 365daysafe from ViroSafe is a specially developed disinfectant and cleaner designed for daily use in aspirator/suction systems. With its unique formulation, it combines powerful antimicrobial action with effective cleaning - all in a single product.

365daysafe is engineered with a low-foaming formulation, ensuring it works seamlessly through suction lines without generating excess bubbles that can interfere with system operation. This makes it safe and efficient for everyday use, while still providing high-level cleaning and disinfection.

Designed for busy dental practices, 365daysafe gets the job done quickly while removing organic residues that can lead to blockages and foul odours. With a 15-minute contact time, it effectively neutralises bacteria, yeast, enveloped viruses and even hard to kill non enveloped viruses such as Norovirus and Adeno.

Unlike many competing products, 365daysafe contains EDTA, which actively dissolves calcium and mineral deposits that naturally accumulate inside suction pipes. This prevents blockages and keeps suction systems running at peak performance. Importantly, the formula is gentle and will not damage or degrade internal pipework making it suitable for daily, long-term use.

While many competitor systems require two separate products; 365daysafe delivers an all in one a single solution. This reduces costs, saves time, and simplifies staff training, all while ensuring consistently high infection control standards.

Instructions for Use Daily.

Add 20ml ± 1ml of concentrate to 980ml warm water (2% solution). Aspirate via the high and low volume suction tubes. Leave for a minimum of 15 minutes and then rinse thoroughly with clean water. For professional use only.

IF SWALLOWED: Rinse mouth. Do NOT induce vomiting. IF INHALED: Remove person to fresh air and keep comfortable for breathing.

Composition.

Aqua, Dimethyldioctylammonium chloride, Dipropylene glycol, 2-Aminoethanol, Tetrasodium ethylene diamine tetraacetate, Ethanol, Citric acid, Nonionic surfactant, Antifoam, Dye.

Storage.

Store in a well-ventilated place. Keep container tightly closed. Keep cool.

Contraindications.

Aluminium.

Disposal.

Do not discharge concentrate into drains or rivers.

Danger. Causes skin irritation. Causes serious eye damage. Very toxic to aquatic life with long lasting effects. Wear protective gloves, protective clothing, eye protection, face protection. IF ON SKIN: Wash with plenty of water. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Immediately call a doctor. If skin irritation occurs: Get medical advice/attention. Collect spillage.

Efficacy and test standards

MICROORGANISM	TIME	CONC %
Bactericidal	15 min	2%
EN13727		
EN14561		
Pseudomonas aeruginosa, Staphylococcus aureus, Enterococcus hirae	15 min	2%
Yeastcidal	15 min	2%
EN13624		
EN14562		
Candida Albicans	15 min	2%
Enveloped Viruses	15 min	2%
EN14476		
EN17111		
Hepatitis B (HBV), Hepatitis C (HCV), HIV, Measles virus, Hepatitis Delta virus (HDV), Influenza virus, Coronavirus, Herpesviridae, Filoviridae (Ebola), Rubella virus, Rabies virus, Poxviridae	15 min	2%
Non-Enveloped Viruses	15 min	2%
EN14476		
EN17111		
Adenovirus 5, Norovirus	15 min	2%



REF

Available in 1 Litre, 2.5 Litre, 5 Litre

VIRS5000

365 daysafe 1 litre

VIRS5001

365 daysafe 2.5 litre

VIRS5002

365 daysafe 5 litre

UFI: J110-E062-M003-J8G9

CE
1639

MD



Manufactured in the United Kingdom by:
Topdental (Products) Ltd. Unit 1, Holmfield
Industrial Estate, Halifax, W. Yorks, HX2
9TN. UK
Contact: info@viroSAFE.com, www.viroSAFEuk.com