

Surfacesafe A+

ALCOHOL BASED SURFACE DISINFECTANT



- Only 60 sec contact time
- Wide effective spectrum
- Powerful action
- Lemon aroma



Ready to use spray designed for the disinfecting of alcohol resistant clinical surfaces.

The powerful alcohol based formula is effective against difficult to kill micro-organisms including Norovirus and Tuberculosis.

Dental clinics demand uncompromising hygiene and reliable protection against even the toughest microorganisms. Surfacesafe A+ from ViroSafe is an alcohol-based surface disinfectant spray formulated for rapid action and superior performance. With its 63% alcohol concentration, Surfacesafe A+ ensures surfaces are safe, streak-free, and ready for use in just moments.

Designed for high-risk clinical environments, Surfacesafe A+ delivers broad-spectrum efficacy in just 60 seconds. Proven effective against a wide range of pathogens, it provides fast-acting protection against bacteria, yeast, enveloped viruses like coronavirus and also those that are more resistant and harder to kill such as Mycobacteria, Norovirus, and Tuberculosis. This ensures confidence in every application and peace of mind for both dental professionals and patients.

Time is critical in a busy practice. Surfacesafe A+ features a rapid evaporation formula that leaves behind a streak-free finish - ideal for hardwearing surfaces including worktops and smooth surfaces where cleanliness and appearance matter. Its quick-drying action also reduces downtime between patients, supporting smooth clinical workflows.

Not all surfaces are suitable for alcohol-based disinfectants, but where they are, Surfacesafe A+ offers outstanding performance. It is ideal for alcohol-resistant materials such as worktops, stainless steel, and equipment cases, ensuring quick and efficient protection without causing damage.

Unlike many alcohol disinfectants with strong, sharp odours, Surfacesafe A+ is enhanced with a light lemon fragrance. This creates a clean, fresh environment for staff and patients without overwhelming chemical smells.

Instructions for Use Daily.

Spray clean surfaces and objects to be disinfected so that they are completely moistened and let the disinfectant take action for 60 seconds. Allow surface spray to evaporate. For professional use only.

Composition.

Ethanol (64-17-5) 63% w/w (630g/kg), Aqua, Perfumes (Contains Linalool, Citral, Geraniol, Alpha-Isomethyl ionone).

Storage.

Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. Store in a well-ventilated place. Keep container tightly closed. Keep cool.

Contraindications.

Alcohol is not recommended for use on dental/medical instruments.
Do not use on surfaces sensitive to alcohol.

Disposal.

Dispose of contents/container in accordance with local regulations.
Do not discharge into drains or rivers.

Danger. Highly flammable liquid and vapour. Causes serious eye irritation. Wear eye protection, protective gloves. If eye irritation persists: Get medical advice/attention. In case of fire: Use alcohol resistant foam to extinguish. Storage. Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. Store in a well-ventilated place. Keep container tightly closed. Keep cool.



Efficacy and test standards

MICROORGANISM	TIME
Bactericidal	60 sec
EN13727	
EN16615	
Pseudomonas aeruginosa, Staphylococcus aureus, Enterococcus hirae, MRSA, VRE	60 sec
Yeasticidal	60 sec
EN13624	
EN16615	
Candida albicans	60 sec
Enveloped viruses	60 sec
EN14476	
Hepatitis B virus (HBV), Hepatitis C virus (HCV), HIV, Measles virus, Hepatitis Delta virus (HDV), Influenza virus, Coronavirus, Herpesviridae, Filoviridae (Ebola), Rubella virus, Rabies virus, Poxviridae	60 sec
Non enveloped viruses	60 sec
EN14476	
Adenovirus 5, Murine norovirus	60 sec
Mycobactericidal	60 sec
EN14348	
Mycobacterium avium	60 sec
Tuberculocidal	60 sec
EN14348	
Mycobacterium terrae	60 sec



REF Available in 1 Litre and 5 Litre

VIRS3001 Surfacesafe A+ 1 litre
VIRS3002 Surfacesafe A+ 5 litre

UFI: EE00-VOC9-F00M-WVDU

CE
1639



Manufactured in the United Kingdom by:
Topdental (Products) Ltd. Unit 1, Holmfield Industrial Estate, Halifax, W. Yorks, HX2 9TN. UK
Contact: info@viroSAFE.com, www.viroSAFEuk.com