

Versaoil

UNIVERSAL OIL SPRAY

- **Effective aerosol based lubricant**
- **High penetration**
- **Use with high or low speed**
- **Safe for pneumatic motors and jointed tools.**
- **Cleans and removes debris in 1-2 seconds.**
- **Pure mint**
- **High pressure penetration with up to 70psi ensures rotation and complete coating of internal bearings.**



Universal nozzle ensures Versaoil fits any dental handpiece

Aerosol based lubricant designed to provide effective lubrication for all types of dental handpieces.

Helps extend handpiece life and avoid malfunction.

High pressure penetration with up to 70psi ensures rotation and complete coating of internal bearings.

Dental handpieces are among the most valuable and frequently used tools in any practice, and protecting them is essential to ensure reliability, precision, and longevity. Versaoil from ViroSafe is a premium aerosol lubricant spray engineered to keep your handpieces operating at peak performance while extending their lifespan.

Versaoil's advanced formulation combines mineral oil, rust inhibitors, silicone, and peppermint oil to deliver comprehensive protection against friction, wear, and corrosion. Regular lubrication reduces mechanical stress on turbines and bearings, helping handpieces run more smoothly while significantly extending their operational life. This means fewer costly repairs and replacements, and more confidence in the reliability of your equipment.

Delivered in a 70 psi aerosol spray, Versaoil ensures powerful penetration into even the smallest internal components. This forceful action guarantees that oil reaches every moving part, providing consistent lubrication and protection where it's needed most. The spray format also simplifies application, making routine maintenance fast, efficient, and hygienic.

Designed with versatility in mind, Versaoil is compatible with all major dental handpieces, including high-speed, low-speed, and air-driven models. This universal performance makes it the perfect all-in-one solution for busy practices seeking both convenience and dependability.

Infused with peppermint oil, Versaoil not only lubricates but also leaves a light, pleasant scent, avoiding the harsh chemical odours found in many traditional sprays. This makes routine maintenance a more pleasant experience for both staff and patients.

Danger. Extremely flammable aerosol. Pressurised container: may burst if heated. Do not spray on an open flame or other ignition source.
Storage. Store in a well ventilated place. Keep cool. Protect from sunlight. Do not expose to temperatures exceeding 50°C/122°F. Keep out of reach of children.
Contraindications. Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking.
Disposal. Do not pierce or burn, even after use.



Instructions for Use.

Insert into back of handpiece and spray through for one to two seconds immediately after each use and before sterilisation. Wipe excess thoroughly. Important: Hold can in a vertical position when in use. Refer to handpiece maintenance instructions where applicable. Use only as directed. For professional use only.

Composition.

Contains white mineral oil with a trace of peppermint oil.

Storage.

Store in a well ventilated place. Keep cool. Protect from sunlight. Do not expose to temperatures exceeding 50°C/122°F. Keep out of reach of children.

Contraindications.

Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking.

Disposal.

Do not pierce or burn, even after use.

REF Available in 500ml e

VIRS9999 Versaoil 500ml
VIRS0025 Universal nozzle

UFI: CW10-G031-P001-5P9X



Manufactured in the United Kingdom by:
Topdental (Products) Ltd. Unit 1, Holmfield
Industrial Estate, Halifax, W. Yorks, HX2
9TN. UK
Contact: info@virosafe.com, www.virosafeuk.com