

INSTRUCTIONS FOR USE

Alcohol Surface Disinfectant

Date of revision: 21/07/25


Available in 5 Litre & 1 Litre dispenser bottles



Intended Use / Intended Purpose:




For the surface disinfection of non-invasive medical devices.
Alcohol is not recommended for use on dental/medical instruments.

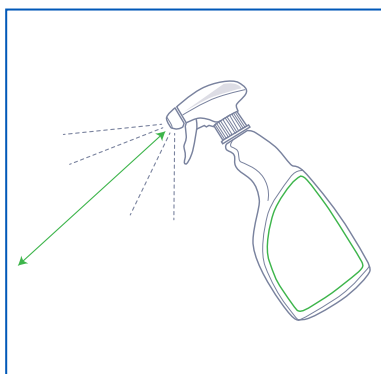

1. Spray clean surfaces and objects to be disinfected so that they are completely moistened.



2. Let the disinfectant take action for 60 seconds.

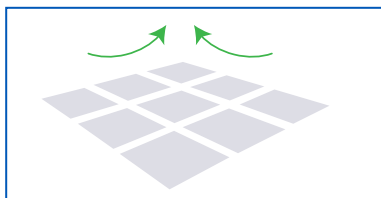

3. Allow surface spray to evaporate. See product labels and MSDS for further information.

Indications / Efficacy (60 secs): Bactericidal including VRE. Yeastocidal. Mycobactericidal. Tuberculocidal. Viruses to include: Non-enveloped viruses: Murine norovirus, Adenovirus 5. Enveloped viruses: Hepatitis B virus (HBV), Hepatitis C virus (HCV), Hepatitis Delta virus, HIV, Influenza virus, Coronavirus, Herpesviridae, Filoviridae (Ebola), Rubella, Rabies, Measles, Poxviridae.

Tested to Standards: EN14885, EN14476, EN14348, EN13727, EN13697, EN13624.

Composition: Ethanol (64-17-5) 63% w/w (630g/kg), Aqua, Perfumes (Contains Linalool, Citral, Geraniol, Alpha-Isomethyl ionone).

Contraindications. Do not use on surfaces sensitive to alcohol.

Disposal. Dispose of contents/container in accordance with local regulations. Do not discharge into drains or rivers.

Intended Users. For professional use only. The products are used within the dental surgery by the dentist or dental assistant as part of their general disinfecting routines.

Patient Population: Dental surgery patients, both adults and children.

UK number for poison advice: 111. Poisons Information Ireland: For information or to report a poisoning incident contact The National Poisons Information Centre (01 8092166). 5L Refill size may only be used to refill dispensers which are labelled with a DAFM approved label for the product.

Any serious incident related to this medical device must be reported to the manufacturer and the competent authorities.

Danger. H225: Highly flammable liquid and vapour. H319: Causes serious eye irritation. P280: Wear eye protection, protective gloves. P337+P313: If eye irritation persists: Get medical advice/attention. P370+P378: In case of fire: Use alcohol resistant foam to extinguish. **Storage.** P210: Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. P403+P233: Store in a well-ventilated place. Keep container tightly closed. P235: Keep cool.



Manufactured in the UK by:
Topdental (Products) Ltd.
Unit 1, Holmfield Industrial Estate,
Halifax, W. Yorks, HX2 9TN. UK
Tel +44 (0)1535 652750
www.topdental.co.uk
www.virosafeuk.com
Email: info@virosafe.com



Danger.



Lemon aroma UFI:EE00-V0C9-F00M-WVDU