

INSTRUCTIONS FOR USE


Daily Aspirator Cleaner and Disinfectant

Date of revision: 17/01/25

Available in 5 Litre / 2.5 Litre / 1 Litre dispenser bottle




Intended Use / Intended Purpose: For the daily cleaning and maintenance of Dental Aspirator Systems.
Can be used on wet line, dry line and semi wet/dry line suction systems

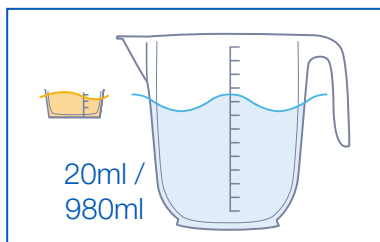


1. Add 20ml $\pm$ 1ml of concentrate to 980ml warm water (2% solution).


The 1L bottle has a built in dosing device and the 2.5L a dosing cap.

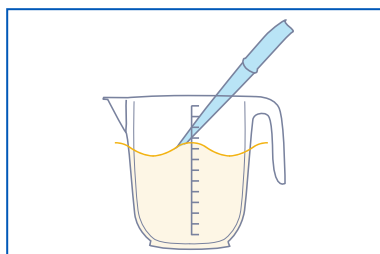


2. Aspirate via the high and low volume suction tubes.

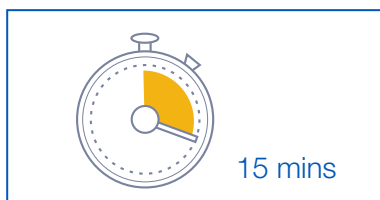

3. Leave for a minimum of 15 minutes.

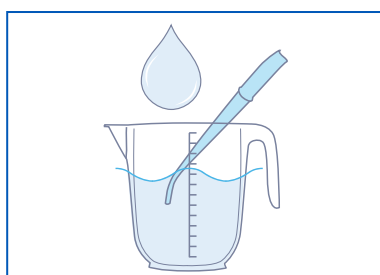

4. Rinse thoroughly with clean water.

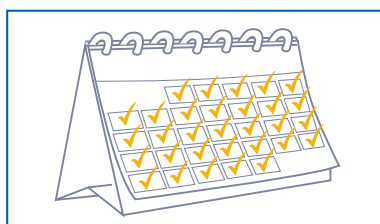

5. Use daily. See product labels and MSDS for further information.

Indications / Efficacy: (15 mins) Bactericidal. Yeastcidal. Viruses to include: Non-enveloped viruses: Murine norovirus, Adenovirus 5. Enveloped viruses: Hepatitis B virus (HBV), Hepatitis C virus (HCV), Hepatitis Delta virus, HIV, Influenza virus, Coronavirus, Herpesviridae, Filoviridae (Ebola), Rubella, Rabies, Measles, Poxviridae.

Tested to Standards: EN17111, EN14885, EN14562, EN14561, EN 14476, EN13727, EN 13624.

Composition: Aqua, Dimethyldioctylammonium chloride, Dipropylene glycol, 2-Aminoethanol, Tetrasodium ethylene diamine tetraacetate, Ethanol, Citric acid, Nonionic surfactant, Antifoam, Dye.

Storage. P403+P233: Store in a well-ventilated place. Keep container tightly closed. P235: Keep cool. **Contraindications.** Aluminium.

Disposal. Do not discharge concentrate into drains or rivers. IF SWALLOWED: Rinse mouth. Do NOT induce vomiting. IF INHALED: Remove person to fresh air and keep comfortable for breathing.

Intended Users. For professional use only. The products are used within the dental surgery by the dentist or dental assistant as part of their general disinfecting routines.

Patient Population: Dental surgery patients, both adults and children.

Any serious incident related to this medical device must be reported to the manufacturer and the competent authorities.

Danger. H315: Causes skin irritation. H318: Causes serious eye damage. H410: Very toxic to aquatic life with long lasting effects. P280: Wear protective gloves, protective clothing, eye protection, face protection. P302+P352: IF ON SKIN: Wash with plenty of water. P305+P351+P338+P310: IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Immediately call a doctor. P332+P313: If skin irritation occurs: Get medical advice/attention. P391: Collect spillage.



Danger.

CE
1639



Manufactured in the UK by:
Topdental (Products) Ltd.
Unit 1, Holmfild Industrial Estate,
Halifax, W. Yorks, HX2 9TN. UK
Tel +44 (0)1535 652750
www.topdental.co.uk
www.virosafeuk.com
Email: info@virosafe.com



UFI: J110-E062-M003-J8G9