


INSTRUCTIONS FOR USE

Impression Disinfectant Powder


Available in 700g




Intended Use / Intended Purpose: Used to disinfect all kinds of impression material, including silicone, alginates and polyether.

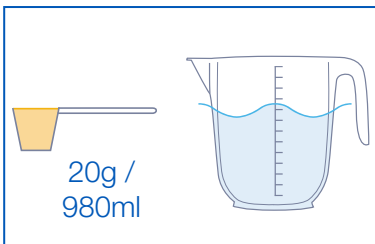


1. Dissolve 1 level measure of powder ($20\text{g} \pm 2\text{g}$) in 980ml of warm tap water ($30 - 40^{\circ}\text{C}$).

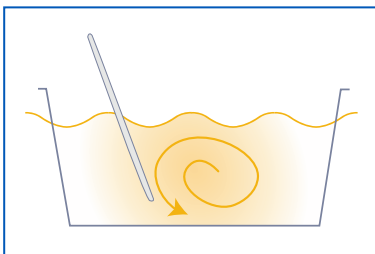


2. Mix for 60 seconds.

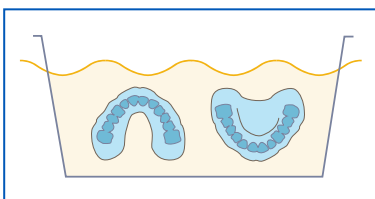

3. Place the impressions into the solution and fully immerse.

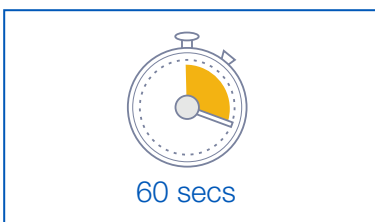

4. 60 second contact time. Do not exceed this recommended time.

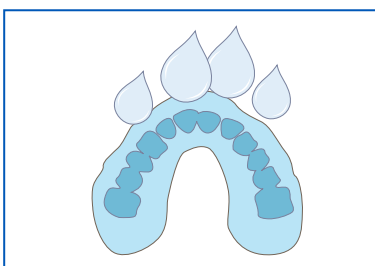

5. Rinse immediately with copious amounts of running water. Change solution daily. See product labels and MSDS for further information.



Indications / Efficacy (60 secs): Bactericidal including MRSA. Yeasticidal. Viruses to include: Non-enveloped viruses: Murine norovirus, Adenovirus 5. Enveloped viruses: Hepatitis B virus (HBV), Hepatitis C virus (HCV), Hepatitis Delta virus, HIV, Influenza virus, Coronavirus, Herpesviridae, Filoviridae (Ebola), Rubella, Rabies, Measles, Poxviridae.

Tested to Standards: EN 17111, EN14885, EN14476, EN13727, EN13697, EN13624.

Composition: TAED, Sodium Percarbonate, Citric Acid Monohydrate, Sodium Alkyl Aryl Sulphonate, Sodium Sulphate Anhydrous, Tetrasodium (1-hydroxyethylidene) bisphosphonate, Sodium Nitrite.

Contraindications. Aluminium, Copper.

Disposal. Do not discharge into drains or rivers.

Intended Users. For professional use only. The products are used within the dental surgery by the dentist or dental assistant as part of their general disinfecting routines.

Patient Population: Dental surgery patients, both adults and children.

Any serious incident related to this medical device must be reported to the manufacturer and the competent authorities.

Danger. H318: Causes serious eye damage. H272: May intensify fire; oxidiser. H412: Harmful to aquatic life with long lasting effects. P280: Wear eye protection, protective gloves. P305+P351+P338+P310: IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Immediately call a doctor. **Storage.** P210: Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. P220: Keep away from clothing and other combustible materials. Store in a well-ventilated place. Keep container tightly closed. Keep dry.



Manufactured in the UK by:
Topdental (Products) Ltd.
Unit 1, Holmfield Industrial Estate,
Halifax, W. Yorks, HX2 9TN. UK
Tel +44 (0)1535 652750
www.topdental.co.uk
www.virosafeuk.com
Email:info@virosafe.com



Danger.

UFI: GF10-X0N2-400J-UYDM