

INSTRUCTIONS FOR USE


Universal Handpiece Oil

Date of revision: 16/01/25


Available in 500ml



Intended Use / Intended Purpose: Handpiece Lubricant Spray for maintenance of all dental handpieces including highspeed, slow speed and air motors. Contains white mineral oil with a trace of peppermint oil.

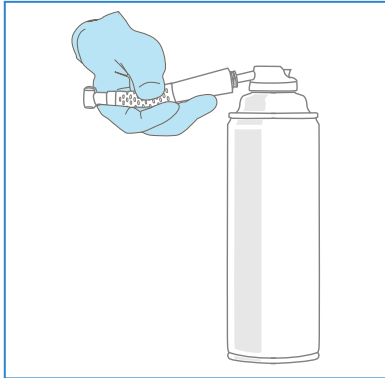


1. Insert into back of handpiece. Use immediately after each use and before sterilisation. Refer to handpiece maintenance instructions where applicable.

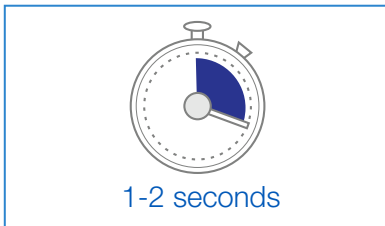


1-2 seconds

2. Spray through for one to two seconds.

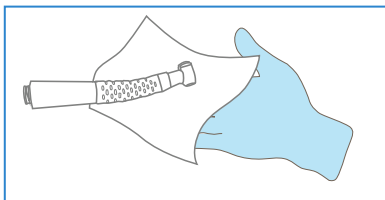

3. Wipe excess thoroughly.

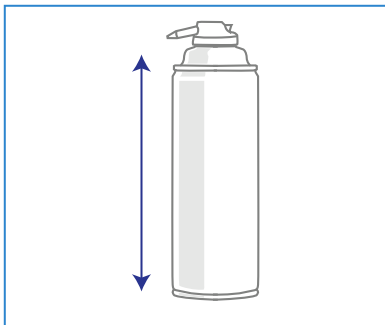

4. Hold can in a vertical position when in use. For professional use only. See product labels and MSDS for further information.

Danger. H222: Extremely flammable aerosol. H229: Pressurised container: may burst if heated. P211: Do not spray on an open flame or other ignition source.

Storage. P403+P235: Store in a well ventilated place. Keep cool. P410+P412: Protect from sunlight. Do not expose to temperatures exceeding 50°C/122°F. P102: Keep out of reach of children.

Contraindications. P210: Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking.

Disposal. P251: Do not pierce or burn, even after use.

Intended Users. For professional use only. The products are used within the dental surgery by the dentist or dental assistant as part of their general disinfecting routines.

Patient Population: Dental surgery patients, both adults and children.

Any serious incident related to this medical device must be reported to the manufacturer and the competent authorities.



Danger.



Manufactured in the UK by:
Topdental (Products) Ltd.
Unit 1, Holmfield Industrial Estate,
Halifax, W. Yorks, HX2 9TN. UK
Tel +44 (0)1535 652750
www.topdental.co.uk
www.virosafeuk.com
Email: info@virosafe.com



UFI: CW10-G031-P001-5P9X